


Test Strips 25 Units

EAN: 8425402
247761

FABRICANTE: ALKANATUR

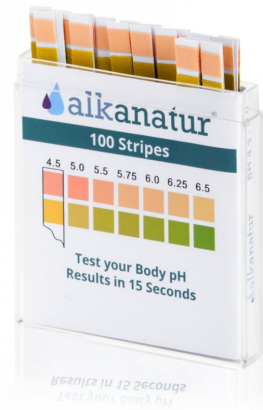

Set of 25 strips for pH measurement.

DESCRIPTION

Body pH test strips

25 units

It is recommended **to measure the pH either through urine or saliva, always on an empty stomach** and the ideal is to do it two or three days in a row and take the average.

It is recommended to dip the test strip for just 15 seconds, either in saliva or urine, and compare it with the color scale that comes in the test strip box. The color that most closely matches the color scale on the box is the actual measurement.

The **ideal pH value would be between 6.75 and 7.25 in urine and between 7.0 and 7.5 in saliva**.

color scale:

