

## Large Menstrual Cup

EAN: 8437009517956

FABRICANTE: GENUINE



*The Genuine menstrual cup is made entirely of sanitary silicone, so it is completely hypoallergenic and adapts completely to your anatomy, providing you with up to 12 hours of maximum protection without discomfort and without noticing that you are wearing it. It is very easy to use. use, it allows you to do sports, even swimming with it on.*

### DESCRIPTION

Size 1 Small (S): women who have not given birth vaginally

Size 2 Large (M): women who have given birth vaginally.

### Instructions for use

#### How do you use the menstrual cup?

Sterilize the cups before and after each menstruation. Wash your hands carefully to avoid infections since the cups are inserted inside the vagina. Fold the cup and insert it. It can be done sitting, standing, squatting or kneeling. Below are the two possibilities in which it can be folded to facilitate insertion. Remove it in a maximum of 8-12 hours, empty the contents and clean it in water. Reinsert the cup.

### What is a menstrual cup?

For all women who want a much more life...COMFORTABLE - SAFE - MODERN - SIMPLE - ECOLOGICAL - ECONOMIC - HYGIENE The menstrual cup is a medical silicone container that is inserted inside the vagina (just like a tampon) for menstruation, adapting to the vaginal walls at all times regardless of the movement we make, and collects all the flow inside the cup. You can wear the cup inside for up to 12 hours without having to worry (depending on your flow). And it has unlimited use, since you can remove it and put it on as many times as you want. It is used daily throughout the cycle. Yes, yes the ENTIRE cycle, so you can use it at night without problems, you can sleep peacefully and peacefully. Then, you just sterilize it with boiling water and store it in its cotton bag until your next menstruation. With it you eliminate for life the tampons, the pads, the panty liners, the odors, the stains, the losses, the worries...

