

## FFP2 MASK BLACK NR UD

**EAN:** 8425402731994    **FABRICANTE:** TEELMASK



*Mask FFP2 Teelmask. Black colour. Suitable to help protect against COVID19 and Monkeypox. Filtration mask. Composition: 100% Polypropylene. respiratory protection against particles, solid and liquid aerosols. 1 unit in each paper container.*

### DESCRIPTION

This mask is made with 4 layers of non-woven fabric:

- Outer Layer - Polypropylene Spunbond
- Second layer - Meltblown Polypropylene
- Third layer - Hydrophobic Polypropylene Spunlace
- Inner layer - Polypropylene Spunbond

It has 1 metal nose clip 5 mm wide, and 88 mm long.

It does not have an exhalation valve.

It has a latex-free elastic strap on each side, 165 mm long.

The minimum particle filtration rate is 95%

It has a low resistance to inhalation, since its action is mechanical and electrostatic.

The expiration date is 2 years from the date of manufacture printed on the packaging.

Complies with the European Harmonized Standard EN 149:2001+A1:2009.

CE0370.

### Instructions for use:

1. Open the paper bag and take out the mask.
2. Place the mask on the face, completely covering the nose and mouth. The nose clip should fit over the nose.
3. Hold the mask with one hand on your chin. Pull the tape out and place it behind your ear.
4. Adjust the nose clip to your nose with both hands, making sure that the edges of the mask are in direct contact with your face.
5. Do a fit-leak test to check that you have put on the mask correctly. Cover the mask with 2 hands and exhale forcefully. If you detect leaks around the edges, reposition the mask on the face, readjust the nose clip, or adjust the tension of the headgear. Repeat this process until no leak is detected.





## FFP2 MASK BLACK NR UD

**EAN:** 8425402731994

**FABRICANTE:** TEELMASK



---

### Attention:

- NR equipment should not be used for more than one work shift. Maximum 8 hours.
- Use under the manufacturer's recommendations, following the instructions on the label.
- This is not a medical device. Not sterile.
- These masks do not provide oxygen.

### Caution:

- Do not use in atmospheres with low oxygen concentration or poorly ventilated areas.
- The presence of special facial features, abundant hair or glasses may prevent a correct fit of the mask.
- Keep away from heat sources.

Product made in Spain.

Prices valid except for changes on our website. (IVA/VAT) included. Doc. Version bf-trf-G-20241130  
<https://bio-farma.es/buy/f/teelmask/ffp2-mask-black-nr-ud>

<https://bio-farma.es> - [info@bio-farma.es](mailto:info@bio-farma.es)

C/ Vermell, 20 3r 3a - 17252  
St. Antoni de Calonge (Girona) Spain



+34 696 79 77 77

